APFD 200

de	Gebrauchsanweisung Einsatz
da	Brugsanvisning Indsats
en	Operating instructions Insert
fi	Käyttöohje Teline
fr	Mode d'emploi Complément
it	Istruzioni d'uso Inserto
nl	Gebruiksaanwijzing Inzet
no	Bruksanvisning Innsats
sv	Bruksanvisning Insats

de 4
da 15
en 26
fi 37
fr 48
it 60
nl 71
no 82
sv 93

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Notes about these instructions	26
Symbols and conventions used in this document	26
Definition of terms	26
Appropriate use	27
Queries and technical problems.....	27
Items supplied	28
Load carriers	28
Disposal of the packing material	28
Optional accessories.....	28
Warning and Safety instructions	29
Areas of application	30
Colour coding.....	30
Preparing the load	31
Preparing the insert	31
Pre-cleaning air regulators	31
Dismantling air regulators.....	31
Inserting air regulators.....	32
Connecting the hose	33
Completely loading the insert.....	34
Filling the pressure chamber	34
Placing the insert in the load carrier	35
After reprocessing	35

en - Notes about these instructions

Symbols and conventions used in this document

Warnings

 Warnings contain information which is important for safety. This alerts you to the potential danger of injury to people or damage to property.

Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Notes provide information of particular importance that must be observed.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

■ Select an option using the arrow buttons and save your choice with *OK*.

Example:

■ Select an option.

Display

Display text can be identified from the special font.

Example:

Save.

Definition of terms

Cleaning machine

In these operating instructions, the dishwasher or the washer-disinfector is referred to as “the machine”.

Load items

The term “load items” is used wherever the items to be processed are not defined in any further detail.

Wash water

The term “wash water” is used for the mixture of water and process chemicals.

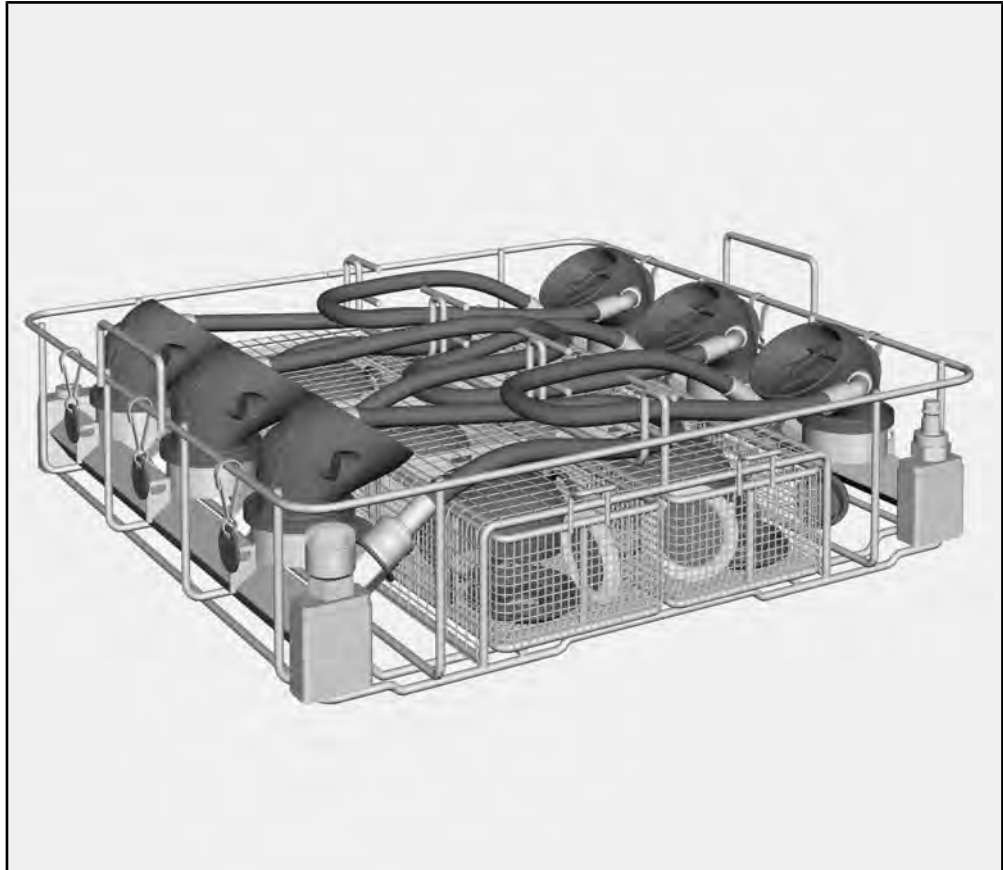
The APFD 200 insert can be used to reprocess machine-reprocessable air regulators in a Miele dishwasher or a machine. Follow the operating instructions for your dishwasher or the machine as well as the instructions issued by the manufacturer of the air regulators.

The APFD 200 insert can be used together with the A 101 upper basket or the A 151 lower basket.

Queries and technical problems

If you have any queries or technical problems that you would like to discuss please contact Miele. Contact details can be found at the back of the Operating Instructions for your washer-disinfector or go to www.miele-professional.com.

Load carriers

- APFD 200 insert, height 153 mm, width 484 mm, depth 452 mm

Items supplied

- 2 x compartment for small parts, divided into three parts
- 6 x carabiner hooks and hangers in 6 colours

Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in landfill sites.

Optional accessories

The following and other accessories are available to order from Miele:

- APFD 202, 6 adapters for loading specially shaped air regulators, e.g. from Interspiro.

en - Warning and Safety instructions

Read the operating instructions carefully before using this insert.
Keep these operating instructions in a safe place.

 Please also read the operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.

- ▶ The inserts are only approved for use in the applications stated in the operating instructions. Applications that go beyond those listed and use of the inserts with additional components from the Miele range of components are described in the operating instructions of the components or are to be agreed with Miele.
Conversions, modifications and any other use are not permitted.
- ▶ Prior to being used for the first time, new inserts must undergo thorough cleaning in the washer-disinfector without load items.
- ▶ Inspect all mobile units, baskets, modules and inserts as described in the “Maintenance” section of the operating instructions for your machine.
- ▶ Only items which have been declared by their manufacturer as suitable for machine reprocessing may be processed. The manufacturer's specific reprocessing instructions must be observed.
- ▶ The last rinse, as a minimum, must be carried out with DI water.
- ▶ Disposable items must not be reprocessed.

en - Areas of application

The APFD 200 insert for reprocessing air regulators must be supplied with pressure for reprocessing. The pressure ensures that wash water cannot enter the air regulators during reprocessing, assuming the correct adapters are used.

The compressed air must correspond to the quality of breathing air as a minimum. For this, observe the national and regional regulations.

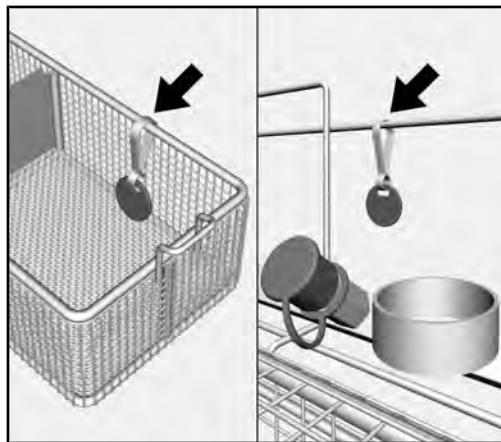
In addition, a flexible compressed air connection must be provided on-site. The compressed air connection must be equipped with one of the following quick-release couplings:

- CEJN series 341
- Parker Rectus series 25S

Colour coding

The small parts of a dismantled air regulator are reprocessed in a compartment of the small-part basket. Colour-code the compartments and the ring holders or the load item carriers to ensure that the small parts can be assigned to the right air regulators after reprocessing.

Coloured tags (2 in each colour) are provided with the inserts for this purpose.


For each colour, attach one tag to the ring holders or the load item carriers and one tag to the small-part basket compartments.

When using other systems for identification, ensure that these are suitably chemical-resistant and temperature-resistant.

Preparing the load

Preparing the insert

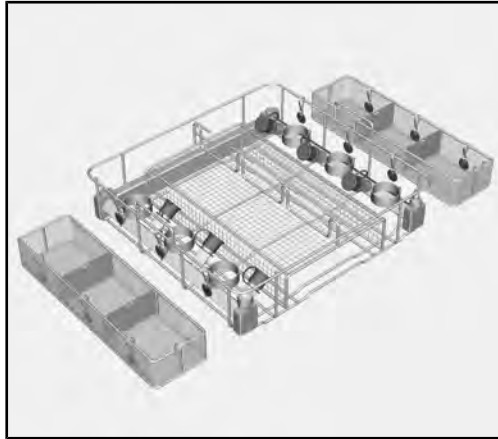- Place the APFD 200 insert on the worktop. The insert must be stable.

- Remove the small-part baskets and place these on the left and right next to the APFD 200 insert.

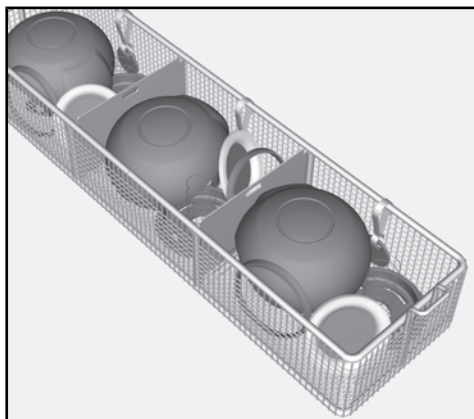
The quick-release couplings on the insert should have the same colour coding as the accompanying compartments of the small-part baskets so that you can assign the air regulator small parts clearly.

Pre-cleaning air regulators

- Check whether the air regulator requires manual pre-cleaning, e.g. to remove soot.
- If required, carry this pre-cleaning out in line with the manufacturer's instructions.

Dismantling air regulators

- Dismantle the air regulator in line with the manufacturer's instructions.


- Place the small parts in a compartment in a small-part basket.
- Do not position small parts in such a way that they will collect water. For example, insert rubber covers with the curved hollow side facing down.

Note the colour-coding for the small-part basket compartment to place the air regulator in the same colour in the insert.

en - Areas of application

Inserting air regulators

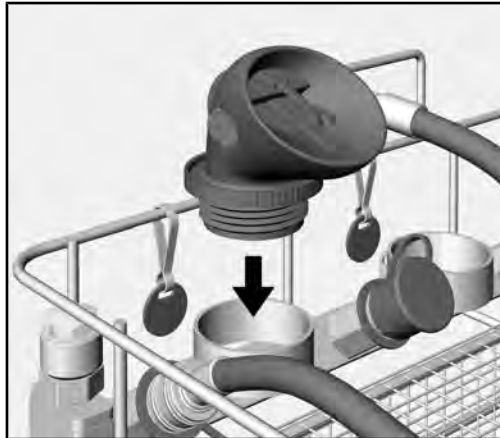- Select the ring holder with the same colour-coding as on the small-part basket.

- Insert the coupling of the air regulator with the opening facing downwards in the ring holder.

For specially shaped air regulators, the ring holders must be equipped with the APFD 202 adapters. The adapters extend the ring holders and ensure a better grip.

The APFD 202 adapter is available as an optional accessory.

Connecting the hose

To supply the air regulator with compressed air, the hose must be connected to the insert.

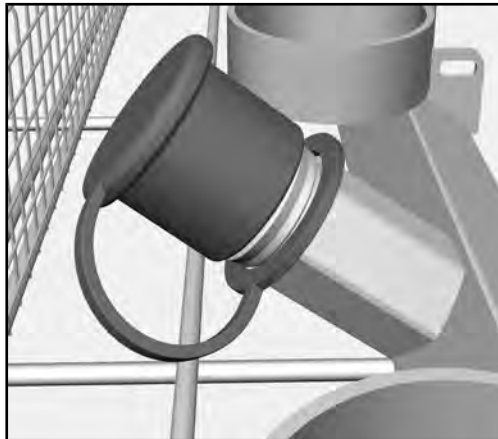
Quick-release couplings

The quick-release couplings on the insert are fitted with protective caps. The protective caps prevent wash water from collecting in the couplings.

If wash water collects in the quick-release couplings, then the pressure can force this into the air regulator when it is connected.


Unused quick-release couplings must therefore always be sealed with protective caps.

- Select the adapter with the same colour-coding as on the ring holder.

- Remove the protective cap from the quick-release coupling.

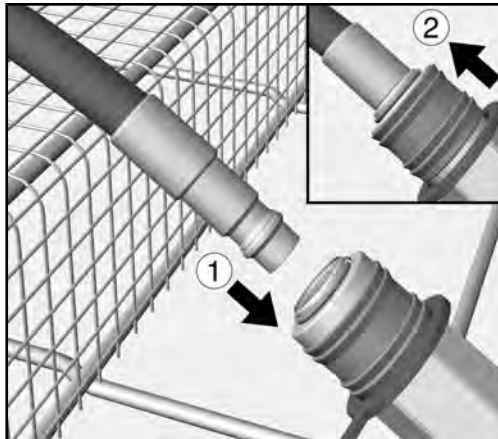Connecting the adapter


- Insert the hose adapter into the quick-release coupling on the ① insert.

The collar on the coupling is pushed downwards in the process. ②. As soon as the collar has snapped back, the hose is pressure-tight.

en - Areas of application

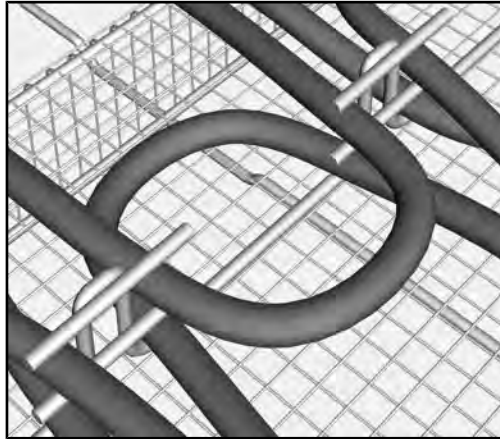Securing the hose

- Lay the hose in an arc shape in the hose guide for the insert.

The locking pins on the hose guide hold the hose in position.

If hoses are not secured, they may block the spray arms during reprocessing.

This will affect the reprocessing result.

Secure all hoses using the locking pins on the hose guide.

Completely loading the insert

- Equip the remaining positions of the insert in the same way.
- Then put the small-parts baskets back in the APFD 200 insert.

Filling the pressure chamber

⚠ Risk of injury due to compressed air.

The jet of compressed air could be dangerous for the user.

Users must be trained on how to use the compressed air system on the customer's premises.

The compressed air must correspond to the quality of breathing air as a minimum. For this, observe the national and regional regulations.

The pressure chamber is designed for pressure ranging from 400–600 kPa.

- Connect the compressed air hose on the customer's premises to the quick-release coupling on the pressure chamber of the insert.
- Provide at least 400 kPa of pressure to the pressure chamber.

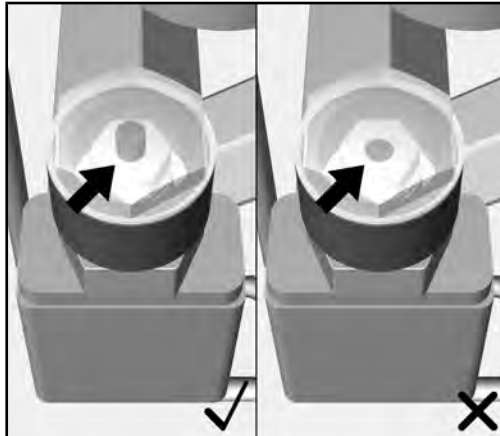
⚠ Damage to the air regulators caused by pressure.

Do not provide more than 600 kPa of pressure to the pressure chamber. High pressure can damage the air regulators.

Observe the air regulator manufacturer's instructions.

Compressed air indicator

The compressed air indicator on the pressure chamber shows whether a sufficient amount of pressure has been provided.

- ✓ If the red indicator of the compressed air indicator pokes out, the pressure is sufficient.
- ✗ If the indicator does not poke out, the pressure is not sufficient.

If the connection between the quick-release couplings on the pressure chamber and the hoses for the air regulators is not be absolutely tight, the pressure will drop over time.

If the pressure drops too much, the wash water may enter the air regulators.

Therefore, supply the pressure chamber with pressure as close to the programme start as possible.

Placing the insert in the load carrier

- Lift the insert at the side handles.
- Place the insert in the load carrier of the dishwasher or the machine.

After reprocessing

It is essential that you pay attention to the “Application technology” chapter in the operating instructions for the dishwasher or the machine and the instructions specified by the manufacturer of the air regulators.

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